			NO.
Instructions This form is to be used by Project Manager, Contractor and Service Engineer. Information is used to develop and determine site ready date. Items listed are go/no go items for delivery unless noted as delay only items. Items identified with *** as delayed items must be completed after hours or on weekend. These items cannot be accomplished while installation is in progress. Also, these items must be completed within two days of installation start or they may stop installation.			CONSTRUCTION
Site Readiness Checklist Customer site preparation verified in general against the Philips final planning drawings. Walls finished including painting. Doors installed. Floor leveled according to Philips drawings and specifications.		I Radiology O RENOVATIONS	ISSUED FOR C
Floors are Illed/covered finished. Flooring is covered with protective covering (scratch protection). Ceiling lights installed. Cable conduit and ductwork installed and clean. Position checked. Duct covers in place but not finally closed. Cable opening are clear, without sharp edges. Pull strings in conduit. Installation per Philips specifications. HVAC environmental equipment installed and working according to Philips specifications. Ceiling installation completed. Electrical preparation according to Philips specifications. All network cabling, drops installed according to Philips specifications (including hardcopy cameras).	PR CONSTRUCTION DOCUMENTS.	Sramhal -LUOR	PORTLAND, ME
All pre-cabling identified on Philips drawings has been installed. Pre-move survey completed - Delivery route identified. Lead glass installed ***. X-ray warning lights installed ***. Dedicated phone line for modem use ***.	Project EasyDiagnost Eleva Maine Medical Center Portland, ME - Room 6 IRUED AS ARCHITECTURAL DRAWNIGS IRUED AS ARCHITECTURAL DRAWNIGS IRUED AS ARCHITECTURAL DRAWNIGS		
Room has been cleaned ***. Cabinets and casework installed***. Modality Checklist Unistrut installed and level according to Philips Specifications.	Erwin Eas Mai lips.com Port Port Port Port Port Port Port Port		
Blocking support for Wall Bucky. Wall support for Wall Bucky. Conduit lengths measured according to Philips specifications. Note: Specifications are from source box to destination box (not just conduit run length). Floorplates installed and level according to Philips specifications.	ager. Wayne riber. 207-65 exerwin@phi.		
All cover plates have holes punched, nipples required, and bushings installed. Approved for Delivery Project Manager	Details 3 Number 100611A	GRAPHIC SCALE: O" SCALE: PROJECT MANAGER: JC/DRAWN BY:	R:
Service Engineer Date	CHK Drawing N.EAS: Oude: Order: Or	A/E OF RECORD: PROJECT NO: DATE:	

Jand, ME DOM 6 AS ARCHITECTURAL DRAWINGS OR CONSTRUCTION DOCUMENTS. Isss in which the equipment is to be installed, used, or stored.		
Bramhall Radiology FLUORO RENOVATIONS	ology	

NOTE: THIS DRAWING WAS NOT PREPARED BY SMRT. IT IS PROVIDED FOR COORDINATION AND REFERENCE PURPOSES ONLY.

Q- 012 ©COPYRIGHT 2008 SMRT INC