

Drawing Index

These sheets are a document set and should not be separated. Electrical information and references are contained on all sheets.

SITE READINESS	C1
EQUIPMENT LAYOUT (Equipment locations, heat loads, component weights, environmental specs)	A1
STRUCTURAL LAYOUT (Structural support/mounting locations for floor/wall/ceiling, wall support elevations)	S1
STRUCTURAL DETAILS (Floor and Ceiling loading information)	S2
ELECTRICAL LAYOUT (Contractor supplied wiring, interconnect methods, junction point locations and descriptions)	E1
ELECTRICAL SPECIFICATIONS (Maximum wiring run lengths, interconnect diagram, system power specifications)	E2
ELECTRICAL DETAILS	E3 THRU E4
MECHANICAL LAYOUT (Chiller information)	M1
EQUIPMENT DETAILS	D1 THRU D3

These drawings indicate the placement and location of the listed equipment and components. The listed equipment components and their locations are prepared for the site. The customer is responsible for ultimately preparing the site for the operation of this equipment in accordance with GE Healthcare's compliance with all applicable federal, state, and/or local requirements.

*** REQUIRED REFERENCE ***


Discovery MR750w
Pre Installation Manual
5670003

A mandatory component of this drawing set is the GE Healthcare Pre Installation manual. Failure to reference the pre installation manual will result in incomplete documentation required for site design and preparation.


Pre Installation documents for GE Healthcare products can be accessed on the web at:

www.gehealthcare.com/siteplanning

GE Healthcare

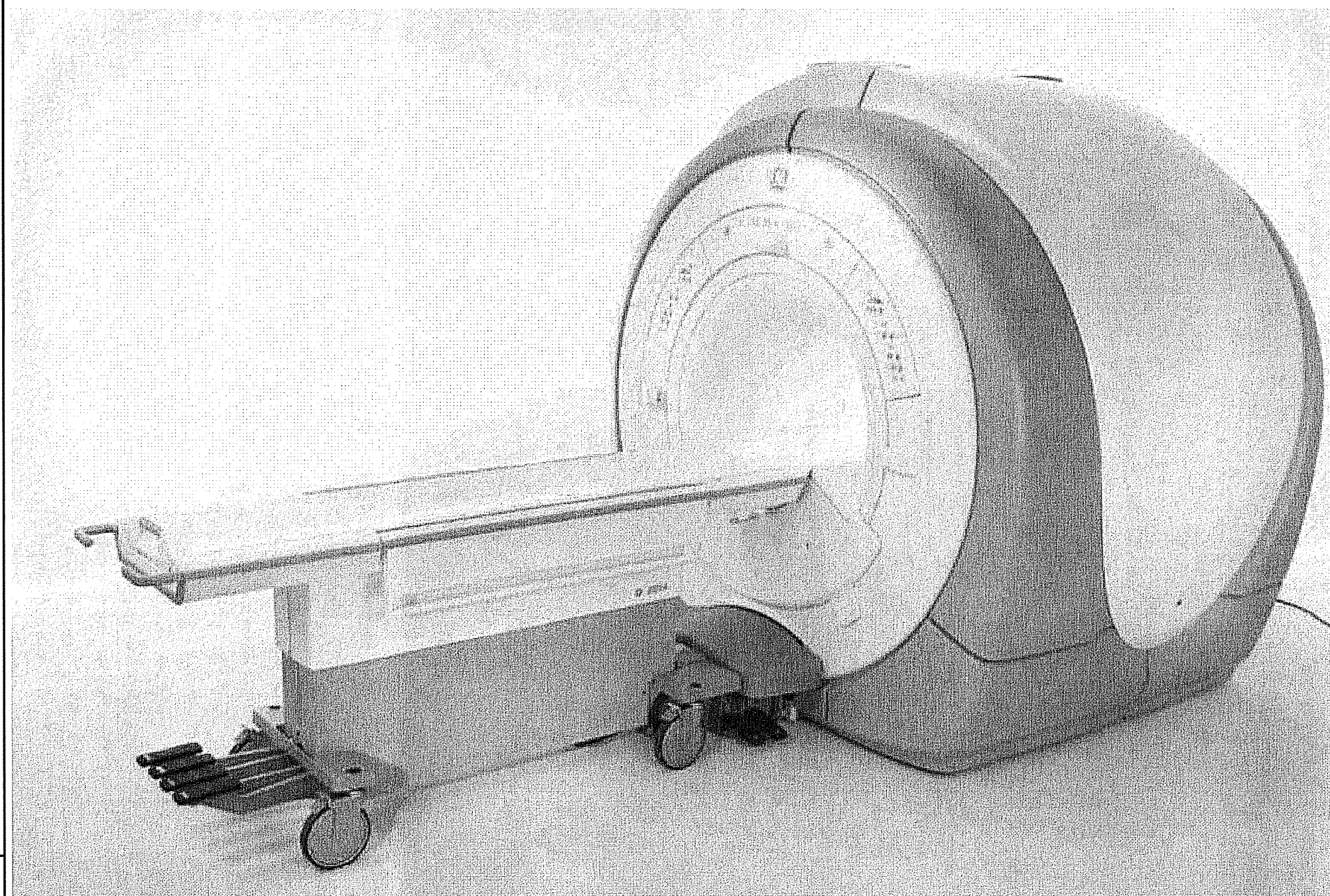

MRi Site Planning

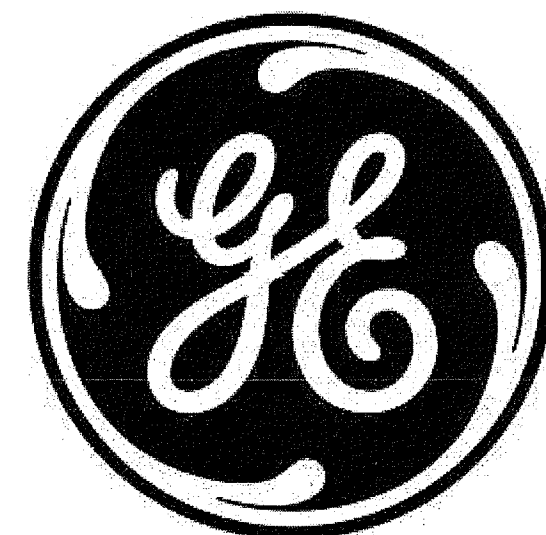

imagination at work

Customer Site Readiness Requirements

- Any deviation from these drawings must be communicated in writing to and reviewed by your local GE Healthcare Installation Project Manager prior to making changes.
- Make arrangements for any rigging, special handling, or facility modifications that must be made to deliver the equipment to the installation site. If desired, your local GE Healthcare Installation Project Manager can supply a reference list of rigging contractors.
- New construction requires the following; 1. Secure area for equipment, 2. Power for drills and other test equipment, 3. Capability for image analysis, 4. Restrooms.
- Provide for refuse removal and disposal (e.g. crates, cartons, packing)
- It is the customer's responsibility to contract a vibration consultant/engineer to implement site design modifications to meet the GE vibration specification. Refer to the system preinstallation manual for the vibration specification.

GE Equipment Delivery Requirements

The items on the GE Healthcare Site Readiness Checklist are REQUIRED to facilitate equipment delivery to the IS site. Equipment will not be delivered if these requirements are not satisfied.

GE Healthcare Site Readiness Checklist Rev 19					
Before using this document ensure you have the latest Rev from MyWorkshop on D000422752					
GEHC Global Order #:		Customer:			
GEHC PMI:		FE / Installer:			
The customer is responsible for proper site preparation regardless of any GEHC measurements/inspections/assessments.					
Inspection Date:					
GEHC Minimum Requirements	Storage in room ready?	PM in room ready?	FE in room ready?	Comments	If "N", enter comments or action plan
1 MR Magnet Delivery Requirements: Ensure cryogen venting system is available for magnet connection as defined by GEHC Pre-Installation Manual (PIM) requirements, exhaust fan system is installed and operational, 480V power, and chilled water supply is available 24/7 that meets system cooling requirements. External connectivity is available for magnet monitoring and phone service is available during delivery. Surface mount/vibromat installed where required. Magnet room final flooring is in place.					
2 MR RF Screen Room Requirements: RF Screen Room is tested with copy of Test Report, emailed to ISAdminCOEMB@ge.com, that it is compliant with GEHC specifications. Dock Bolt and magnet anchors (if applicable) installed using 2 part anchor. For HDx systems, blower box mount bolts installed by RF vendor using 2 part anchors.					
3 State Regulatory Requirements: Facility registration number provided for states of IL, KY, HI, RI, SC, TX. X-ray shielding plan and state acknowledgment letter provided to installer for AR, DC, NC, SC, CO, S.W.A.					
4 Site Drawing Requirements: Final version of equipment network and antenna, installation drawings (including red lined versions) verified to match actual room and has been provided to installer.					
5 Surface Penetration Requirements: Customer/Contractor scheduled to provide required drilling or cutting into floors, ceilings, and walls; OR surface penetration permit available and posted in the room when GEHC will perform the work.					
6 Pre-Delivery Route Requirements: The equipment delivery route from the truck to the final destination within the facility has been reviewed with all key stakeholders to safely meet the minimum requirements for equipment access, and all communications/notifications have occurred. Arrangements have been made for special handling (elevator, rigging, floor protection, fork lift, rollback truck, etc).					
7 Finished Room Requirements: Rooms that will contain equipment, including storage areas not in scan suite, are dust free. Provisions taken to maintain a dust free room. Precautions must be taken to prevent dust from entering rooms containing equipment when construction is incomplete in adjacent areas. All walls primed (final coat not needed on Day 1). Shielding doors, and windows are to be installed. No contractor work being done during or after the installation that will cause dust in the installation areas or potential equipment damage. Room security to prevent unauthorized access and theft has been discussed with customer. The customer is aware of these security issues, implications and responsibility. For Storage: Room must meet PIM requirements for storage.					
8 Electrical Requirements: Lockable (LOTO) Main Disconnect Panel (MDP) is installed per GE guidelines and system power is available. Conduits, electrical cable ducting/dividers/cable trays, and access flooring is installed in proper location and height. Surface floor duct and local-side wires can be installed at time of system installation. Validate outlet location and requirements meet specifications for device/equipment.					
9 HVAC Requirements: The HVAC/Chilled Water systems designed to maintain the environment per spec/PIM is at running state and apps to provide the desired environmental conditions including location of vents, temperature and humidity for system operation.					
10 Flooring Requirements: Floor is clean and prepared for final floor covering. Floor levelness/flatness is measured and within tolerance, and there are no visible defects per GEHC specifications. Confirm customer anchoring plan aligns with designed floor thickness. Final flooring installed where required for network racks.					
11 Ceiling Requirements: Unistrut (or equivalent) location, levelness and spacing is measured (or vendor confirmed) and consistent with the requirement of the installation drawings. Ensure unistrut and rails are not used as mounting surfaces. Ceiling grid is installed. Permanent lighting is installed and operational. HVAC diffusers are installed and connected to ductwork. Ceiling tiles installed per PIM discretion.					
12 Staging Requirements: Space has been identified to support the active installation process only. This area meets PIM/project book requirements.					
13 Storage space has been identified, if needed. This secured space would be used to store equipment indefinitely. If offsite, transportation plan has been developed at customer expense. This space must meet PIM requirements.					
14 Network Connectivity: Hardwire for network connectivity(network drop) is in place prior to delivery with specified network firewall configuration where required. Site Surveys for wireless mobile XR units have been completed.					
15 Medical Gases Requirements: Systems (hard piped or portable) in place to allow testing and calibration of equipment (anesthesia), including ventilation.					

GE Healthcare
 GE Healthcare is a division of General Electric Company. Design Center
 Milwaukee, Wisconsin
 Copyright 2009 General Electric Company - Proprietary to GE

SHEET TITLE: **SITE READINESS**
 MODALITY TYPE: **DISCOVERY MR 750w GEM**
 THIS PLAN IS SUBMITTED TO SUGGEST LOCATION OF GE HEALTHCARE EQUIPMENT AND ASSOCIATED ELECTRICAL, MECHANICAL, AND ROOM REQUIREMENTS. IT IS NOT TO BE USED FOR ACTUAL CONSTRUCTION PURPOSES. HOWEVER, AND THE COMPANY CANNOT ACCEPT RESPONSIBILITY FOR ANY DAMAGES RESULTING THEREFROM.

PROJECT TITLE:
MAINE MEDICAL CENTER
 PORTLAND, MAINE

PROJECT: 132054
 REVISION: 01
 DATE: 02.Jul.13
 DRAWN BY: MCK
 CHECKED BY: JGA
 GON NO: 4108424
 GON DT: 27.Jun.13

REVISION HISTORY:
 1 KMR - 26.Aug.13
 CHECKED BY: PMM

SHEET
C1

This drawing is based on Sketch No.: 13me045
 PIM R4
 RQ - 137547