

EQUIPMENT DETAIL WORKSTATION

M1013AW
REV. DATE: 04/25/01

ULTRASPARC STATION
21" COLOR MONITOR
KEYBOARD
FLAT PANEL MONITOR

DETAIL NOT TO SCALE

EQUIPMENT DETAIL UPS SYSTEM (CAT. NO. B7864PP)

B7864PP
07/27/05

PLAN VIEW
FRONT VIEW
SIDE VIEW

DETAIL NOT TO SCALE

EQUIPMENT DETAIL POWER DISTRIBUTION UNIT

B78-58D
REV. DATE: 09/18/03

PLAN VIEW
FRONT VIEW
SIDE VIEW

NOTE:
• INDICATES AIR FLOW
• INDICATES CENTER OF GRAVITY

EQUIPMENT DETAIL OPERATORS CONSOLE

B78-58A
REV. DATE: 05/17/05

PLAN VIEW
FRONT VIEW
SIDE VIEW

NOTE:
• INDICATES AIR FLOW
• INDICATES CENTER OF GRAVITY

SEISMIC BRACKET PROVIDED WITH CONSOLE			
DIM	MIN.	MAX.	
"A"	26.77" [680mm]	34.76" [883mm]	
"B"	26.77" [680mm]	31.77" [807mm]	
"C"	40.18" [1020mm]	48.35" [1228mm]	
"D"	18.77" [428mm]	24.86" [634mm]	

EQUIPMENT DETAIL MAIN LINE CONTACTOR - OPTIONAL

E45-02AE
REV. DATE: 02/20/04

TOP VIEW
FRONT VIEW
SIDE VIEW
BACK VIEW

NOTE:
• DETAIL NOT TO SCALE

EQUIPMENT DETAIL CT GANTRY SHIPPING DETAIL

B78-64D
REV. DATE: 09/20/04

PLAN VIEW
GANTRY DELIVERY SIZE OPTIONS

CONFIGURATION	WEIGHT lb [kg]	LENGTH in [mm]	WIDTH in [mm]	HEIGHT in [mm]	PERSONS NEEDED	ADDITIONAL PERSON HOURS	COMMENT
DOLLIES ON SIDE RAILS ON	4501 [2078]	114 [2900]	51 [1290]	79 [2000]	2	0	VAN DRIVERS DELIVER EQUIPMENT TO ROOM. SEE REPS. SUPERVISE AND ASSIST. HEIGHT IS WITH 1 IN. (25.4mm) CLEARANCE FROM FLOOR.
DOLLIES ON SIDE RAILS REMOVED	114 [2900]	32.4 [830]	79 [2000]	10.00" [254mm]	2	0.5	WITH RAILS REMOVED, BE VERY CAREFUL TO PREVENT DAMAGE TO COVERS.
REMOVE SIDE RAILS AND ONE DOLLY				11.00" [279mm]	2	1.5	THIS CONFIGURATION IS USED TO GET GANTRY INTO A 9 FT. (2.74m) ELEVATOR.
GANTRY SET ON AUX. DOLLIES					2	2.5	DO NOT PUSH ON GANTRY COVERS. PUSH ONLY ON FRAME MEMBERS. AUX. DOLLIES PROVIDE 2.3 IN. (59mm) CLEARANCE FROM FLOOR.
REMOVE COVERS - USE SHIPPING DOLLIES					4	1.5	USE EXTREME CARE IN REMOVING, HANDLING AND INSTALLING COVERS. SEE PROCEDURE IN DIRECTION 224312-100, QX/I PLUS GENERAL SYSTEM INSTALLATION. SEE TWO PREVIOUS COMMENTS.
REMOVE COVERS - USE AUX. DOLLIES					4	3.5	
DOLLIES OFF, LOADERS OFF	3834 [1739]	81 [2060]	34 [860]	73 [1850]	4	3.5	SEE LIFTING PROCEDURE DIRECTION 224312-100, QX/I PLUS GENERAL SYSTEM INSTALLATION.

EQUIPMENT DETAIL CT TYPICAL SCATTER SURVEY

B78-64E
REV. DATE: 05/12/05

PLAN VIEW
ELEVATION

NOTE: 140 Kv
100 mA
1 sec. 40 mm aperture
LARGE PHANTOM

EQUIPMENT DETAIL CT GANTRY AND TABLE LAYOUT

B78-64A
REV. DATE: 02/10/05

PLAN VIEW

ALLOW FOR ADEQUATE SERVICE CLEARANCE
• CENTER OF GRAVITY

EQUIPMENT DETAIL CT GANTRY AND TABLE LAYOUT

B78-64B
REV. DATE: 12/15/04

SIDE VIEW

EQUIPMENT DETAIL CT GANTRY

B78-64C
REV. DATE: 05/17/05

FRONT VIEW
SIDE VIEW

SHIPPING DIMENSIONS:
114"(2895mm)L x 51"(1295mm)W x 79"(2006mm)H
• CENTER OF GRAVITY

EQUIPMENT DETAIL CT TYPICAL SCATTER SURVEY

B78-64E1
REV. DATE: 05/12/05

PLAN VIEW
ELEVATION

NOTE: 140 Kv
100 mA
1 Sec. 4 x 5.00mm
HEAD PHANTOM

EQUIPMENT DETAIL GT 2000 TABLE

B79-20
REV. DATE: 12/15/04

PLAN VIEW
SIDE VIEW

• CENTER OF GRAVITY

GE Healthcare Technologies
Installation Services Design Center
Milwaukee, Wisconsin

SHEET TITLE: EQUIPMENT DETAILS
MODALITY TYPE: LIGHTSPEED VCT

THIS PLAN IS SUBMITTED TO SUGGEST LOCATION OF GE HEALTHCARE EQUIPMENT AND ASSOCIATED APPARATUS, ELECTRICAL WIRING DETAILS AND ROOM ARRANGEMENTS. IT IS NOT TO BE USED FOR ACTUAL CONSTRUCTION PURPOSES. HOWEVER, THE COMPANY CANNOT ACCEPT RESPONSIBILITY FOR ANY DAMAGES RESULTING THEREFROM.

PROJECT TITLE:
**MERCY HOSPITAL
FORE RIVER
PORTLAND, MAINE**

PROJECT NO: 065779
REVISION: 00

DATE: 09-22-06
DRAWN BY: AJS
CHECKED BY: REK
QUOTE NO: KX1CB6
QT. DATE: 09-07-06

REVISION HISTORY:

SHEET
D1